



THIS IS MY LOOK

Small tweaks. Dramatic changes.

Or even nothing at all.

How you define your look is up to you.

This material is intended for distribution by healthcare professionals to patients prescribed or recommended with Restylane® treatment.

Restylane®
by GALDERMA

DEFINE YOUR OWN LOOK

THE AGING PROCESS

- As we age, our skin loses fat, collagen, and other important elements that provide facial volume and support
- We also start to see the formation of lines, wrinkles, and folds that can contribute to us looking older

DEFINE YOUR OWN LOOK

- It's not only about smoothing the lines, wrinkles, and folds that come with age. Restylane products can also be used to help you achieve your personalized signature look by creating lift, structure, and volume

THE IMPORTANCE OF HYALURONIC ACID (HA)

- HA is a molecule that is found in the body and plays an important role in hydrating our skin and providing fullness and elasticity
- However, our natural levels of HA decline as we age which can cause skin to lose volume and increase the chances of wrinkles and skin folds



RESTYLANE PRODUCTS CONTAIN A CLEAR GEL FORMULATION OF HA THAT IS **SPECIFICALLY DESIGNED TO ACT LIKE YOUR BODY'S OWN NATURALLY PRODUCED HYALURONIC ACID TO GIVE YOU THE MOST INDIVIDUALIZED AND NATURAL-LOOKING RESULTS.**¹⁻⁶

FOR BESPOKE, NATURAL-LOOKING RESULTS



HIGH PERFORMANCE THAT LASTS

Fillers from the Restylane range achieve results that **last up to 24 months**, depending on the product used and area treated^{*7,8}



THE CLOSEST TO NATURAL HA

Closest to your skin's own HA for **individualized and natural-looking results**²⁻⁵



A TRUSTED PRODUCT

Restylane pioneered the original HA filler over 26 years ago and has been used in **more than 55 million treatments worldwide**, in a diverse range of skin colors, tones, and types^{**†9-12}



ENHANCES YOUR NATURAL EXPRESSION

Restylane products work to enhance your face's natural movement for **youthful-looking results**¹³



A HIGH LEVEL OF TREATMENT SATISFACTION

99% of patients would recommend a Restylane treatment to a friend¹⁴



* A second treatment (retreatment) was performed at 12 months in the same areas with the same products. Touch-up was allowed 1 month after both treatments.⁷

** Restylane injectable gel was first approved for marketing and sale in September 1996 in the EU.⁹

† Based on a Restylane publications analysis conducted in June 2021. Sources include a published post-marketing safety study of 638 patients and a retrospective analysis of adverse event reports encompassing 406,000 patients.¹²

ACHIEVE THE LOOK YOU WANT WITH RESTYLANE® RANGE OF FILLERS¹⁻⁸

- ① TEAR TROUGHS
- ② TEMPLES
- ③ CHEEKS
- ④ NOSE
- ⑤ NASOLABIAL FOLDS
- ⑥ PERIORAL LINES
- ⑦ MARIONETTE LINES
- ⑧ LIPS
- ⑨ JAWLINE
- ⑩ CHIN



EACH **RESTYLANE FILLER IS SPECIFICALLY DESIGNED**
TO WORK IN A DIFFERENT AREA OF THE FACE, MEANING
YOUR TREATMENT CAN BE **PERSONALIZED TO YOUR**
INDIVIDUAL NEEDS^{1-6,9}



ENHANCE YOUR SKIN QUALITY WITH RESTYLANE® SKINBOOSTERS™

- ① CHEEKS
- ② CHIN
- ③ PERIORAL AREA
- ④ PERIOCCULAR AREA
- ⑤ FOREHEAD
- ⑥ TEMPLES
- ⑦ NECK
- ⑧ HANDS
- ⑨ DÉCOLLETAGE



RESTYLANE SKINBOOSTERS - THE ORIGINAL AND
LONG-LASTING SKIN BOOSTING INJECTABLE - **ENHANCE**
6 KEY ASPECTS OF SKIN QUALITY WHILE PROVIDING
DEEP HYDRATION¹⁻⁹



TWO COMPLEMENTARY TECHNOLOGIES

TO HELP ACHIEVE **THE LOOK YOU WANT**^{1,2}

NASHA[®]

A FIRM GEL FOR PRECISE AND PRONOUNCED LIFTING AND STRUCTURE, WITHOUT ADDING VOLUME^{*,**1,3,4}

RESTYLANE[®]

RESTYLANE[®]
LYFT[™]

OBT[™]

A SOFTER GEL FOR TAILORED CONTOURING AND NATURAL VOLUME^{1,5-7}

RESTYLANE[®]
REFYNE[™]

RESTYLANE[®]
KYSSE[™]

RESTYLANE[®]
VOLYME[™]

RESTYLANE[®]
DEFYNE[™]

Your doctor will select the **appropriate Restylane product, depending on your individual needs**, to help you achieve the look you want.

THE VERSATILITY OF RESTYLANE TECHNOLOGIES IS DESIGNED TO GIVE YOU **INDIVIDUALIZED, NATURAL-LOOKING RESULTS THAT LAST**^{1,4,8}

* Product integration was assessed in an ex vivo human skin model using histological methods.⁴

**The firm gel structure of NASHA prevents distribution into the surrounding tissue after injection, enabling targeted product integration and precision.⁴

NASHA, Non-Animal Stabilized Hyaluronic Acid; **OBT**, Optimal Balance Technology.

RESTYLANE® LYFT™ ULTIMATE PROJECTION

DESIGNED TO LIFT AND **ACCENTUATE YOUR FEATURES TO
GIVE YOU STRUCTURE** WITHOUT VOLUME^{1,2,9}

A **firm gel** texture that's ideal
for enhancing your **cheeks, nose,
chin, and jawline**^{5,9,10}


Designed to be durable, with
results lasting up to 24 months^{*11}

CHEEKS

JAWLINE

NOSE

CHIN

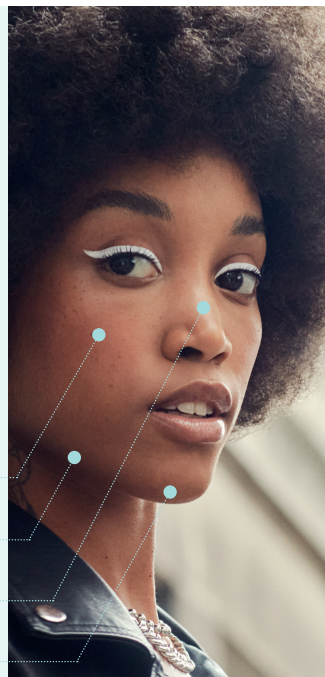

**PATIENT
SATISFACTION:**



99%

of patients **would
recommend the
treatment to a friend**¹²

96%

of patients felt their
treatment **met or
exceeded expectations**¹²

* At 24 months 93% of patients (N=100) had self-evaluated improvements and 82% had physician-assessed improvements following one retreatment.¹²

RESTYLANE® KYSSE™ KISSABLE LIPS

DESIGNED FOR **SOFT, PLUMP, AND
NATURAL-LOOKING LIPS**¹⁻⁴



FOR EVEN MORE KISSABLE LIPS^{2,3,5}

A **soft, flexible and supportive** gel texture that has the ability to integrate with your tissue for **natural-looking, kissable lips**^{2,3,5}

It plumps, softens, and naturally enhances your lips, **restoring shape and improving lip texture**^{2,3,5}

For durable results **lasting up to 12 months**^{*6}

LIPS

PATIENT AND PARTNER SATISFACTION:



100%

of patients **were satisfied** or **very satisfied** with their lip style after treatment^{**2,6}

9/10

partners **liked the way** their partners lips **looked and felt**^{†2}

* Optional retreatment was offered at 12 months and a maximum total volume of 3 mL was permitted. Touch-up was allowed after 2 weeks.⁶

** As assessed by patients and investigators for both upper and lower lips (lips rated as "improved", "much improved", or "very much improved"), (N=31).⁶

† Partners who rated "very satisfied", "satisfied" vs. "neutral", "unsatisfied" or "very unsatisfied", (N=59).²

RESTYLANE® VOLYME™ EFFORTLESS VOLUME

SPECIFICALLY DESIGNED TO CREATE A **FULLER,**
NATURAL-LOOKING, MORE YOUTHFUL LOOK⁷⁻¹²

A GEL FORMULATION THAT DELIVERS NATURAL-LOOKING VOLUME⁹⁻¹⁴

The **soft and flexible gel** is specifically designed to restore the natural roundness and softness of your face to give you a **youthful look** while allowing **natural facial movement**⁸⁻¹²

Durable enough to retain shape and volume over time, with **volumizing effects maintained for up to 18 months**⁹

TEMPLE

CHEEKS



PATIENT SATISFACTION:



100%

of patients **would recommend the treatment**⁹

>98%

of patients were **satisfied** or **very satisfied** with the durability of the results^{*9}

* As assessed by subject satisfaction questionnaires. Patients who reported that they were "satisfied" or "very satisfied", at 18 months after the last injection (N=60).⁹

RESTYLANE® DEFYNE™ DESIGNED TO RESTYLANE® REFYNE™ ENHANCE YOUR NATURAL EXPRESSION

**THE FIRST HA FILLERS CLINICALLY PROVEN TO RESTORE
A YOUTHFUL EXPRESSION¹⁻³**

SUBTLE DEFINITION

Achieve subtle definition with **Restylane Defyne** – used to contour and define your face and chin²⁻⁷

NASOLABIAL FOLDS

MENTAL CREASE

CHIN



Treatment lasts
up to 12 months^{*1,11}

**PATIENT
SATISFACTION:**

97%



of patients **were satisfied** with their treatment, 1 month after treatment^{**12}

FLEXIBLE REFINEMENT

Give your face flexible refinement with **Restylane Refyne** – used to fill lines and wrinkles^{2,7-10}

PERIORAL LINES

NASOLABIAL FOLDS

MARIONETTE LINES



Treatment lasts
up to 12 months^{*1,11}

**PATIENT
SATISFACTION:**

95%



of patients **would recommend** the treatment to a friend, 1 month after treatment^{**13}

* Touch-up was allowed at 2 weeks after initial treatment.¹²

** Pooled results for both Restylane Refyne and Restylane Defyne. As assessed by subject and investigator satisfaction questionnaires.^{12,13}


HA, hyaluronic acid.

DEFYNE YOUR CHIN TO LOOK YOUR BEST FROM EVERY ANGLE

WHAT DOES CHIN RETRUSION LOOK LIKE?



No retraction

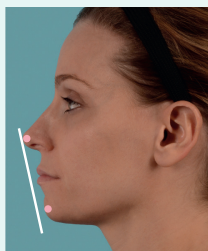

Mild retraction



Moderate retraction



Substantial retraction

RESTYLANE DEFYNE OFFERS AN IDEAL BALANCE OF **FIRMNESS AND FLEXIBILITY**, TO CONTOUR AND **DEFINE THE CHIN**, AND GIVE YOU: ^{*4,6-8}

- FLEXIBLE FACIAL MOVEMENT
- NATURAL EXPRESSIONS

PATIENT SATISFACTION:

99%

of patients said the **appearance of their chin had improved** after treatment with Restylane Defyne^{**14}


DURATION



* HA-based filler products have been tested for firmness (G') and flexibility (xStrain) using rheometry. G' value reflects the gel firmness (support) of a product. xStrain is an index of gel flexibility (how much strain a gel can withstand and still be reversible).⁷

** As assessed by patients and treating investigators on the 7-point GAIS (patients who were scored as "improved" "much improved" or "very much improved") after 12 weeks, (N=107).¹⁴

RESTYLANE® VERSATILE SUPPORT

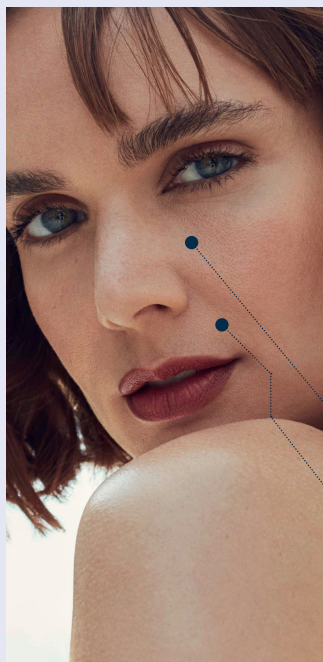

A **VERSATILE OPTION** THAT CAN BE USED TO **ACHIEVE THE LOOK YOU WANT**¹⁻⁴

The **original hyaluronic acid filler** with more than two decades of real-world experience^{*5,6}

Durable results that last for **up to 36 months**^{**7}

Ideal to reduce the appearance of **nasolabial folds, marionette lines, and infraorbital hollows**¹

INFRAORBITAL HOLLOWS

NASOLABIAL FOLDS & MARIONETTE LINES

PATIENT SATISFACTION:



100%

of patients **reported improvements** at 14 days after treatment^{†8}

>96%

of patients felt that they **would do the treatment again**^{†9}

* Restylane injectable gel was first approved for marketing and sale in September 1996 in the EU.⁵

** With one retreatment.⁷

† As evaluated by subject-rated improvement in the GAIS (ratings: "very much improved", "much improved", "improved" vs "no change", "worse"), (N=60).⁸

‡ As measured by subject satisfaction questionnaires regarding treatment conducted at timepoints throughout the study, (N=100).⁹

GAIS, Global Aesthetic Improvement Scale.


RESTYLANE® BRIGHTER LOOK

A **TARGETED TREATMENT** TO
REDUCE THE APPEARANCE OF
UNDER-EYE HOLLOWNESS AND
**REVITALIZE YOUR UNDER-EYE
AREA**^{1,10,11}

Restylane helps **fill in your
under-eye hollows** to improve
the appearance of under-eye
shadows and to give you a
brighter appearance¹⁰

Treatment with Restylane offers
improvements that **last up to 6
months**^{1,10}

INFRAORBITAL HOLLOWS

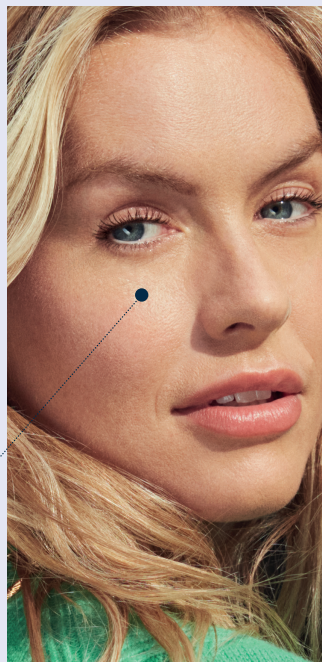

**PATIENT
SATISFACTION:**



94%

of patients **would repeat
the treatment again** and
recommend the treatment
to a friend^{*10}

83%

of patients were
“satisfied” or
“extremely satisfied”
with their treatment^{**10}

* Conducted as a post-study survey, (N=35).¹⁰

** 82.9%. As measured by the patient-evaluated 5-point SSQ (global satisfaction ratings: “extremely satisfied”, “satisfied”, “neutral”, “dissatisfied”, “extremely dissatisfied”).¹⁰

SSQ, subject satisfaction questionnaire.

RESTYLANE® SKINBOOSTERS™

THE ORIGINAL AND ONLY **SKIN QUALITY INJECTABLE**
CLINICALLY PROVEN TO **ENHANCE 6 KEY ASPECTS**
OF SKIN QUALITY¹⁻⁹

UNMATCHED PERFORMANCE:

ENHANCING 6 KEY ASPECTS
OF SKIN QUALITY³⁻⁹



GLOW FROM WITHIN WITH RESTYLANE SKINBOOSTERS^{9,10}



CLINICALLY PROVEN TO BOOST SKIN QUALITY FOR SMOOTH, EVEN, YOUNGER-LOOKING, AND GLOWING SKIN³⁻⁹

- If you have **dry, dull, uneven or crêpey skin**, you could benefit from treatment with Restylane Skinboosters³⁻⁹



FAST-ACTING, IMMEDIATE, DEEP AND LONG-LASTING HYDRATION^{3,9}

- Restylane Skinboosters uses the natural ability of HA to bind water molecules, locking moisture within the skin^{1-3,9}



LONG-LASTING DEEP HYDRATION AND IMMEDIATE RESULTS THAT LAST FOR UP TO 15 MONTHS³

- Restylane Skinboosters offers **long-lasting skin quality improvements for up to 15 months³**



TRUSTED* WITH OVER 18 YEARS OF CLINICAL EXPERIENCE^{**3-8,12}

- The closest to natural stabilized HA to improve the natural skin structure and skin hydro-balance^{3-6,10,11}



CONSISTENTLY HIGH SATISFACTION^{5,10,13}

- After 12 months of treatment, **91% of patients wanted to repeat their treatment**, and **96% would recommend it** to their friends⁵

SKIN-BOOSTING INJECTABLE
WITH RESULTS THAT LAST
FOR UP TO:

15 months³

BEFORE



AFTER



* In more than 8 Million treatments globally.¹²

** Restylane Skinboosters Vital obtained first CE certification in Europe in 2004.¹²

HA, hyaluronic acid.

YOUR RESTYLANE PERSONALIZED TREATMENT PLAN

DATE	AREA OF TREATMENT

RESTYLANE PRODUCT	NOTES

TREATMENT RESULTS

INDIVIDUAL RESULTS MAY VARY

LORENA, 24 YEARS

BEFORE



AFTER



RESTYLANE®
KYSSE™

LIP
1 ml

RESTYLANE®
DEFYNE™

CHEEKS
1.5 ml/side

TAFAREL, 32 YEARS

BEFORE



AFTER



RESTYLANE®
LYFT™

CHIN
1 ml

JAWLINE
1.3 ml/side

RESTYLANE®
DEFYNE™


PRE-JAW
0.5 ml/side

RESTYLANE®
SKINBOOSTER™
VITAL

SKIN REJUVENATION
1 ml/side

JAQUELINE, 34 YEARS

BEFORE

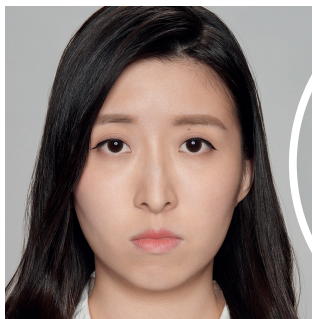

AFTER



RESTYLANE®
DEFYNE™

CHEEKS
0.7 ml/side

RESTYLANE®
LYFT™

CHIN
1.8 ml

REBECCA, 37 YEARS

BEFORE



AFTER



RESTYLANE®
DEFYNE™

NASOLABIAL FOLDS
1 ml/side

RESTYLANE®
KYSSE™

LIP
0.3 ml

RESTYLANE®
DEFYNE™

CHIN
0.4 ml/side

TREATMENT RESULTS

INDIVIDUAL RESULTS MAY VARY

ANAHITA, 32 YEARS

BEFORE



AFTER



RESTYLANE®
LYFT™

CHIN

1 ml

NOSE

1 ml

CHEEKS

2 ml/side

RESTYLANE®
KYSSE™

LIP VOLUME

0.8 ml

YU, 39 YEARS

BEFORE



AFTER



RESTYLANE®

TEAR TROUGH

0.6 ml right, 0.5 ml left

RESTYLANE®
LYFT™

NOSE

0.5 ml

CHIN

0.5 ml center

RESTYLANE®
DEFYNE™

CHIN

0.5 ml right

REFERENCES

PAGES 2-3: DEFINE YOUR OWN LOOK/ FOR BESPOKE, NATURAL LOOKING RESULTS 1. Öhrlund Å. *J Cosmet Dermatol Sci Applic* 2018; 8(2): 47-54. 2. Kablik J et al. *J Dermatol Surg* 2009; 35: 302-312. 3. Edsman K et al. *Dermatol Surg* 2012; 38(7 Pt 2): 1170-1179. 4. Weiss R et al. *Dermatol Surg* 2016; 42: 699-709. 5. Talarico S et al. *Dermatol Surg* 2015; 41: 1361-1369. 6. Philipp-Dormston WG et al. *Dermatol Surg* 2018; 44: 826-832. 7. Huang S and Tsai T. *J Drugs Dermatol* 2020; 19(9): 836-842. 8. Narins R et al. *Dermatol Surg* 2011; 37: 644-650. 9. Galderma. Data on File. MA-39680. 10. Taylor S et al. *Dermatol Surg* 2009; 35: 1653-1660. 11. Kerscher M et al. *Dermatol Surg* 2008; 34: 720-726. 12. Galderma. Data on File. MA-48087. 13. Percec I et al. *Plast Reconstr Surg* 2020; 145(2): 295e-305e. 14. Andriopoulos B et al. Poster presented at Aesthetic & Anti-aging Medicine World Congress. 4-6 April 2019. Monte Carlo, Monaco.

PAGES 4-5: ACHIEVE THE LOOK YOU WANT WITH RESTYLANE® RANGE OF FILLERS 1. Restylane® Lyft™ Singapore IFU. 2. Restylane® Kysse™ Singapore IFU. 3. Restylane® Volyme™ Singapore IFU. 4. Restylane® Refyne™ Singapore IFU. 5. Restylane® Defyne™ Singapore IFU. 6. Restylane® Singapore IFU. 7. Galderma. Data on File. MA-43049. 8. Galderma. Data on File. MA-34483. 9. Gregario G et al. *Clin Cosmet Investig Dermatol* 2022; 15: 681-690.

PAGES 6-7: ENHANCE YOUR SKIN QUALITY WITH RESTYLANE® SKINBOOSTERS 1. Restylane® Skinboosters™ VITAL Singapore IFU. 2. Restylane® Skinboosters™ VITAL LIGHT Singapore IFU. 3. Wu Y et al. *J Cosmet Dermatol*. 2020. 19(7):1627-1635. 4. Williams S et al. *J Cosmet Dermatol*. 2009;8(3):216-225. 5. Gubanova E et al. *J Drugs Dermatol*. 2015;14(3):288-299. 6. Dierickx C et al. *Dermatol Surg*. 2018;44 Suppl 1:S104-S18. 7. Lee BM et al. *Arch Plast Surg*. 2015;42(3):282-287. 8. Ribé A et al. *J Cosmet Laser Ther*. 2011;13(4):154-161. 9. Kim J. *Arch Aesthetic Plast Surg*. 2014;20(2):97-103.

PAGES 8-9: TWO COMPLEMENTARY TECHNOLOGIES TO HELP ACHIEVE THE LOOK YOU WANT AND RESTYLANE LYFT 1. Gregario G et al. *Clin Cosmet Investig Dermatol* 2022; 15: 681-690. 2. Nikolis A et al. *Aesthet Surg J Open Forum* 2020; 2(1): oja005. 3. Huang S and Tsai T. *J Drugs Dermatol* 2020; 19(9): 836-842. 4. Lundgren B et al. *J Drugs Dermatol* 2018; 17(9): 982-986. 5. Galderma. Data on File. MA-43049. 6. Philipp-Dormston W et al. *Dermatol Surg* 2018; 44: 826-832. 7. Talarico S et al. *Dermatol Surg* 2015; 41: 1361-1369. 8. Nikolis A et al. Poster presented at the 20th Aesthetic & Anti-aging Medicine World Congress. 31 March-2 April 2022. Monte Carlo, Monaco. 9. Restylane® Lyft™ Singapore IFU. 10. Öhrlund Å. *J Cosmet Dermatol Sci Applic* 2018; 8(2): 47-54. 11. Huang S and Tsai T. *J Drugs Dermatol* 2020; 19(9): 836-842. 12. Andriopoulos B et al. Poster presented at AMWC. 4-6 April 2019. Monte Carlo, Monaco.

PAGES 10-11: RESTYLANE KYSSÉ AND RESTYLANE VOLYME

1. Restylane® Kysse™ Singapore IFU. 2. Bertucci V et al. *J Cosmet Dermatol* 2021; 20(5): 1499-1504. 3. Bertucci, V et al. *J Drugs Dermatol* 2021; 20: 402-408. 4. Galderma. Data on File MA-43049. 5. Lundgren B et al. *J Drugs Dermatol* 2018; 17(9): 982-986. 6. Hilton S et al. *Dermatol Surg*. 2018; 44(2): 261-269. 7. Restylane Volyme Singapore IFU. 2020. 8. Gregario G et al. *Clin Cosmet Investig Dermatol* 2022; 15: 681-690. 9. Talarico S et al. *Dermatol Surg* 2015; 41(12): 1361-1369. 10. Jones D et al. *J Drugs Dermatol* 2021; 20(9): 949-956. 11. Nikolis A et al. *Aesthet Surg J Open Forum* 2020; 2(1): oja005. 12. Kestemont P et al. *J Drugs Dermatol* 2012; 11(1 Suppl): s9-s16. 13. Segura S et al. *J Drugs Dermatol* 2012; 11(1 Suppl): s5-s8. 14. Rogerio V et al. *J Stomatol Oral Maxillofac Surg* 2022; 123(4): 440-447.

PAGES 12-13: RESTYLANE DEFYNE/REFYNE AND DEFYNE YOUR CHIN TO LOOK YOUR BEST FROM EVERY ANGLE 1.

Philipp-Dormston W et al. *J Cosmet Dermatol* 2020; 19(7): 1600-1606. 2. Solish N et al. *J Cosmet Dermatol* 2019; 18(3): 738-746. 3. Percec I et al. *Plast Reconstr Surg* 2020; 145(2): 295e-305e. 4. Restylane® Defyne™ Singapore IFU. 5. Gregario G et al. *Clin Cosmet Investig Dermatol* 2022; 15: 681-690. 6. Rogerio V et al. *J Stomatol Oral Maxillofac Surg* 2022; 123(4): 440-447. 7. Öhrlund Å. *J Cosmet Dermatol Sci Applic* 2018; 8(2): 47-54. 8. Galderma. Data on File. MA-43049. 9. Restylane® Refyne™ Singapore IFU. 10. Rohrich R et al. *Plast Reconstr Surg Glob Open* 2019; 7(6): e2172. 11. Ascher B et al. *Dermatol Surg* 2017; 43(3): 389-395. 12. Philipp-Dormston W et al. *Dermatol Surg* 2018; 44(6): 826-832. 13. Philipp-Dormston W et al. Poster presented at the Aesthetic & Anti-aging Medicine World Congress. 6-8 April 2017. Monaco. 14. Marcus K et al. *Plast Reconstr Surg* 2022. Online ahead of print.

PAGES 14-15: RESTYLANE 1. Restylane® Singapore IFU. 2. Galderma.

Data on File. MA-43049. 3. Öhrlund Å. *J Cosmet Dermatol Sci Applic* 2018; 8(2): 47-54. 4. Restylane® Injectable Gel. Summary of safety and effectiveness data (SSED). Available at: www.accessdata.fda.gov/cdrh_docs/pdf2/P020023b.pdf (accessed November 2022). 5. Galderma. Data on File. MA-39680. 6. Galderma. Data on File. MA-48087. 7. Narins R et al. *Dermatol Surg* 2011; 37: 644-650. 8. Weiss R et al. *Dermatol Surg* 2010; 36(S1): 750-759. 9. Huang S and Tsai T. *J Drugs Dermatol* 2020; 19(9): 836-842. 10. Nikolis A et al. *Aesthet Surg J* 2022; 42(3): 285-297. 11. Lundgren B et al. *J Drug Dermatol* 2018; 17(9): 982-986.

PAGES 16-17: RESTYLANE SKINBOOSTERS 1. Restylane®

Skinboosters™ VITAL Singapore IFU. 2020. 2. Restylane® Skinboosters™ VITAL LIGHT Singapore IFU. 3. Wu Y et al. *J Cosmet Dermatol*. 2020. 19(7):1627-1635. 4. Williams S et al. *J Cosmet Dermatol*. 2009;8(3):216-225. 5. Gubanova E et al. *J Drugs Dermatol*. 2015;14(3):288-299. 6. Dierickx C et al. *Dermatol Surg*. 2018;44 Suppl 1:S104-S18. 7. Lee BM et al. *Arch Plast Surg*. 2015;42(3):282-287. 8. Ribé A et al. *J Cosmet Laser Ther*. 2011;13(4):154-161. 9. Kim J. *Arch Aesthetic Plast Surg*. 2014;20(2):97-103. 10. Gubanova E et al. *Aesthetic Med* 2010;1:94-98. 11. Edsman K et al. *J Dermatol Surg* 2012;38(7 Pt 2):1170-1179. 12. Galderma. Data on File. MA-39929. 13. Kerscher M et al. *Dermatol Surg* 2008; 34(5): 720-726.

BACK COVER: YOUR RESTYLANE QUESTIONS ANSWERED

1. Hilton S et al. *Dermatol Surg* 2018;44(2):261-269. 2. Weiss RA et al. *Dermatol Surg* 2016;42(6):699-709. 3. Restylane® Singapore IFU. 4. Andriopoulos B et al. Poster presented at AMWC. 4-6 April 2019. Monte Carlo, Monaco.

YOUR RESTYLANE QUESTIONS ANSWERED

WHICH TREATMENT IS RIGHT FOR ME?

Each Restylane product is designed to achieve something different. This means you can work with your doctor to achieve a bespoke look tailored to your needs.

HOW LONG DO THE EFFECTS OF A RESTYLANE TREATMENT LAST?

The effects of Restylane are long-lasting but not permanent. Restylane is biodegradable so it will gradually disappear from the body over time.¹ **Restylane treatments can last up to 24 months***, but this period may vary, depending on the individual, the amount used, and the area being treated. After your initial treatment, follow up sessions are usually recommended to maintain results.^{1,2}

SHOULD I AVOID ANYTHING AFTER TREATMENT?

It's best to avoid exposing the treated area to extreme heat (e.g. sunbathing, saunas, or steam baths) until any signs of inflammation have disappeared.³

ARE THERE ANY TREATMENT SIDE EFFECTS?

The most common post-treatment reactions are skin redness, swelling, itching, bruising, or pain and tenderness at the treatment site. These reactions usually disappear shortly after each injection, within 2 weeks.^{1,2} If these become troublesome or persistent, or if you are concerned about any side effect, contact your doctor.

* At 24 months 93% of patients (N=100) had self-evaluated improvements and 82% had physician-assessed improvements following one retreatment.⁴

Galderma (S) Pte Ltd
1 Kim Seng Promenade #14-10 Great World City
Singapore 237994

SG-RES-2300015

Restylane[®]
by GALDERMA